110-107-400 4m CE

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Silicone return electrode size 8 x 15 cm for Ellmann, Valleylab and Micromed HF- units MD 100 from 12/25, cable length 4 m

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Warning Notes:

- There is a danger of burn injuries if the return electrode is not properly applied.
- Always apply the return electrode in proper relation to the respective operating area.
- Never use contact gel in applying the return electrode.
- Be sure to arrange the silicon return electrodes supply leads in such a way that they will not get in contact with either the patient or other cables or lines. Avoid loops.
- Prior to increasing HF power to a point exceeding normal values, verify that the return electrode is applied properly across its entire contact surface.
- These return electrodes are delivered in hygienically perfect but not sterile packing.
- The return electrode should be applied in an area near the operating area where blood circulation is good. However, always keep this electrode at a minimum distance of 20 cm from the place where the operation is to take place. Moreover, make sure in selecting the electrode application area that no currents will flow across the heart area!
- Critical tissue areas and bony projections are not suitable for applying the return electrode. If the application area selected is covered with hair, shave and clean the area prior to applying the electrode.
- non-compliant with EU standards
- Silicone return electrodes are to be used in veterinary field only

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Instructions for use:

- Observe the Operating Instructions accompanying your high frequency device.
- Carefully clean the application area selected but do not use alcohol! Subsequently dry it fully.
- Do not use contact gel.
- Attach the electrode fully across its entire contact surface and in a fold-free manner.
- Connect the return electrode to the HF device.
- Upon completion of the operation, carefully remove the electrode.
- Can be sterilized in Autoclave up to +134° C.

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Medizintechnik GmbH