170-001-000
Arbeitsselement bipolar, passiv, L= 185 mm
Working element bipolar, passive, L=185 mm

170-002-000
Arbeitsselement bipolar, aktiv, L= 185 mm
Working element bipolar, active, L=185 mm

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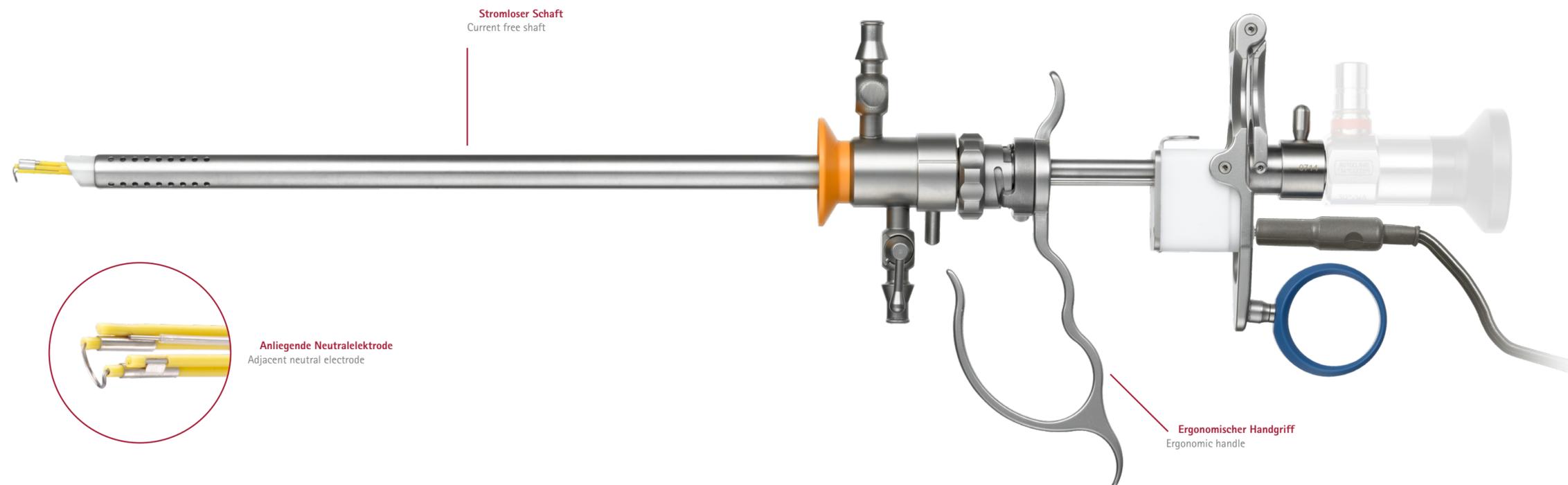
The bipolar ReSekto system greatly reduces the risks associated with the use of conventional monopolar systems, risks such as uncontrolled current flow are eliminated. With ReSekto current flow is limited to the space between the two distally placed electrodes and occurs only in the area visible to the surgeon. As bipolar resection is performed in saline solution, the risk of a TUR syndrome occurrence is practically eliminated. The ReSekto bipolar electrode arrangement ensures outstanding ignition performance, with precise, reliable and consistent cutting and coagulation quality throughout the whole procedure.

This specially designed arrangement of the two distal electrodes also ensures that there is no current flow along the sheath, thus preventing unwanted thermal tissue damage caused by leakage current or damaged insulation. The ReSekto system design allows simple insertion and removal of the electrode. The ergonomic handle, the simple and self-explanatory detachability of the instrument and the cable connector being positioned parallel to the working axis enable safe and efficient working. All components of the ReSekto system are made from high quality materials that easily withstand current cleaning and sterilisation requirements.

- Current free shaft
- No accidental tissue coagulation by second electrode
- Electrical current flow limited to the operating field
- Excellent cutting and coagulation performance due to narrow electrical distance between active and neutral electrode in combination with Micromed electro surgery unit MDV touch 200
- No TUR syndrome risk through use of NaCl
- Easy to replace electrodes
- Generators, optics and sheath independent in the open system
- Reusable electrodes
- Single bipolar connector
- Ergonomic handle

	3 m	130-171-300		3 m	130-121-300
2-Banana plug	5 m	130-171-500	Micromed, Martin, Aesculap	5 m	130-121-500
	4 m	130-331-400		3 m	130-151-300
ERBE VIO 300 D			Valleylab	5 m	130-151-500

	171-002-000 Schlingenelektrode, 90° abgewinkelt Loop electrode, 90° angled		171-014-000 Kugelelektrode Ball electrode		171-012-000 Messerelektrode Knife electrode
	171-004-000 Schlingenelektrode, 60° abgewinkelt Loop electrode, 60° angled		171-006-000 Schlingenelektrode, gerade Loop electrode, straight		171-016-000 Walzenelektrode Roller electrode